EndoProbe® Handpiece Models

ENGLISH INSTRUCTIONS FOR USE

INDICATIONS

This product is indicated for use in ophthalmic laser treatments to deliver laser energy inside the eye. This device is cleared for use for the particular indications of the laser system to which it is attached.

DIRECTIONS FOR USE

- This product contains a fragile fiber-optic cable. Remove probe from package and unwind with care. Excessive stress may damage the fiber optic.
- Connect the probe to an Iridex laser system or another laser system validated for use with this product. Consult your local Iridex representative for compatibility details.
- This product is packaged sterile and intended for single use only. Do not reuse, may lead to patient injury and/or infection.

Aspirating EndoProbe Handpiece

Connect the open luer lock to the aspiration line. Insert the tip of the probe in sterile normal saline and aspirate until the line is free of bubbles.

Illuminating EndoProbe Handpiece

Connect the probe directly, or via an appropriate adapter, to a light source.

WARNING ⚠

- Excessive treatment power may result in a retinal hole or hemorrhage. Excessive power delivered at short pulse durations may result in choroidal hemorrhage.
- Do not use if the pouch has been opened, damaged or is wet.
- Do not open sterile package prematurely. Open sterile package just prior to use to maintain sterility of contents.
- This device must be used with an appropriate laser safety filter. Never look directly into the laser light source or at laser light scattered from reflective surfaces.
- Store package under normal storage conditions - in a dry, clean, well ventilated area at room temperature, between 15°-25°C (59°-77°F).

FEDERAL (U.S.) LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

CAUTION: Federal law restricts this device to sale by or on the order of a physician.

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© 2021 Iridex. All rights reserved. 15447 Rev G 12.2021 1