



1 Power On

- Locate the key switch, insert key and turn clockwise from off to on

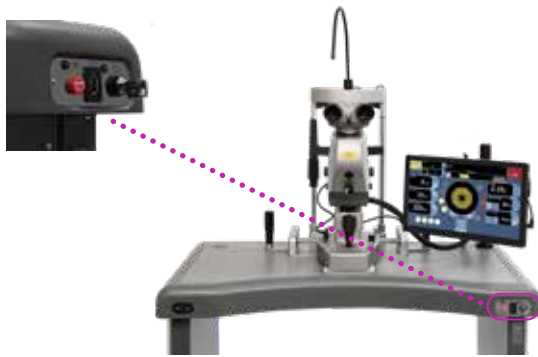

2 Slit Lamp Illumination

- Adjust slit lamp illumination intensity, width and height

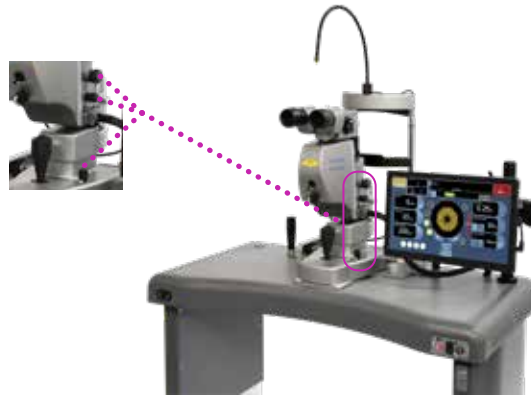


3 Slit Lamp Controls

Micromanipulator

- Adjusts the aiming beam for both single spot and patterns

Power Knob

- Allows surgeon to easily adjust laser power without sitting back from binoculars

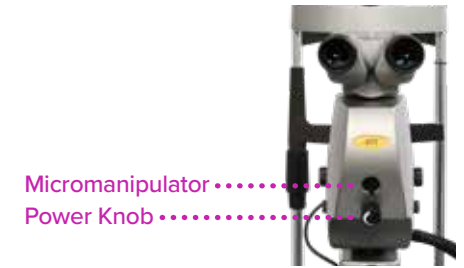

4 Table Adjustment


- Raise PASCAL table up or down with arrow button on left side of the table

Height Adjustment Controls




5 Connect LIO

- Locate the single functional LIO port
- Slide LIO Port cover down to expose LIO port

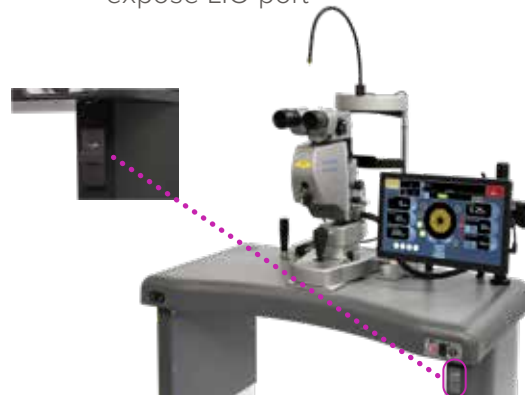

6 Emergency Shut-off

- Press the red button on the front of the laser console to quickly cease laser function

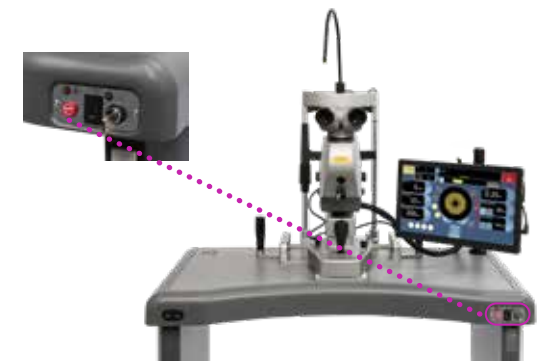



Iridex Corporation
1212 Terra Bella Avenue
Mountain View, California 94043-1824 USA

Telephone: (650) 940-4700
(800) 388-4747 (US only)
Fax: (650) 962-0486
Technical Support: (650) 962-8100
techsupport@iridex.com



The symbols glossary is provided in the Symbols section of the Operator Manual of the Laser console.

This Quick Start guide is a summary document for user convenience, but this quick start guide does not replace the operator manual provided with the Laser System. Do not use or operate the Laser System until you have read and understood the operator manual.