




Quick-Guide: Using the Cyclo G6[®] Laser with Sweep Management Software and the MicroPulse P3[®] Delivery Device*




Turn key to **"ON"**.
Wait for the initialization to complete. Connect MicroPulse P3 Probe.

4 **Setting Treatment Parameters**
Set parameters using the Preset Menu or by manual entry.

Preset Option

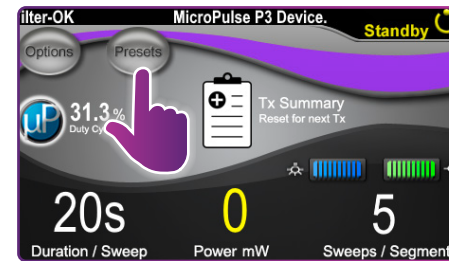

Manual Option

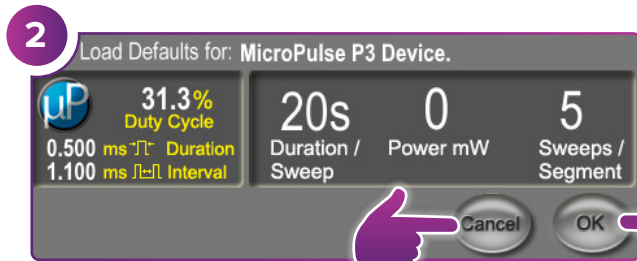


Select **"OK"** to set new parameters. A pop-up screen will appear (Step 3).

Select **"CANCEL"** to retain parameters used in prior treatment. A pop-up screen will appear (Step 3).




SEE MORE STEPS ON BACK PAGE

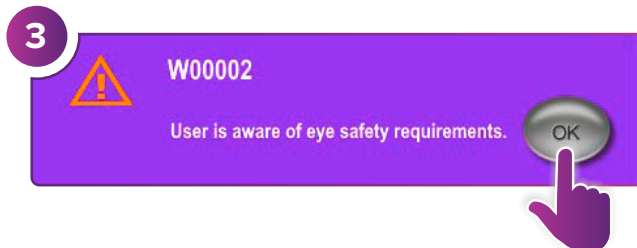


Press **"OK"**.
To set new parameters, go to Step 4.
To retain your parameters, go to Step 5.



* Before delivering treatment, please refer to the "User Guide: Cyclo G6 Software Upgrade for MicroPulse TLT," the "Cyclo G6 Laser System Operator Manual," and the "MicroPulse P3 Delivery Device Instructions for Use" for delivery instructions, including safety, warnings, and cautions.

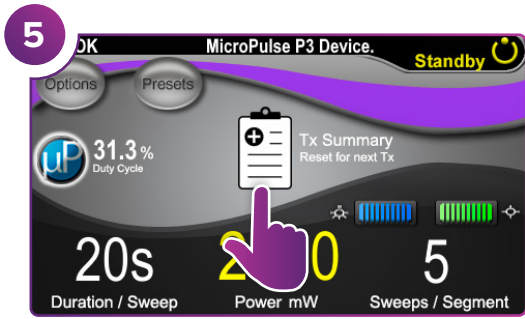


Select "Tx Summary".

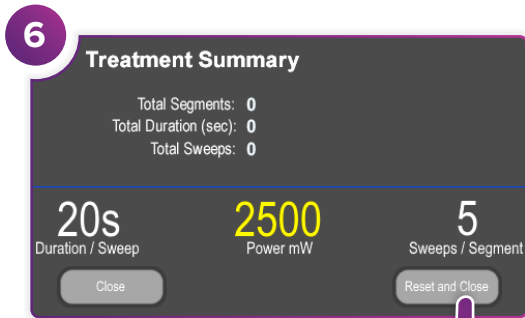


Confirm "Total Segments", "Total Duration" and "Total Sweeps" values are reset to zero.



Press this button to enter Treat Mode (enable laser).

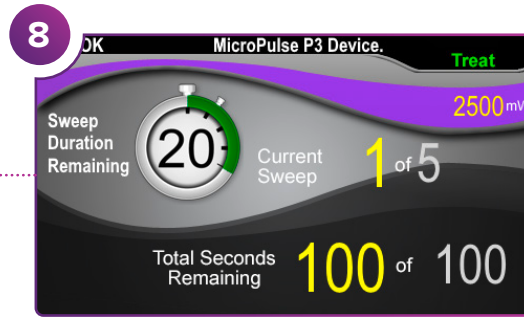


Deliver treatment to desired segment.

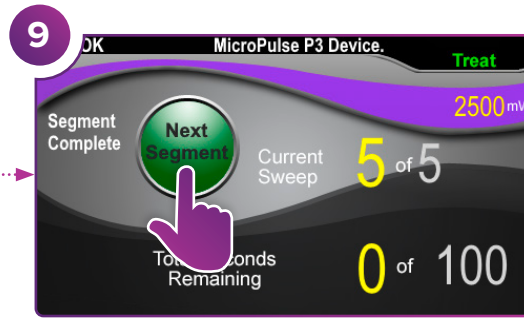


Press to reset values. To treat next segment, repeat Step 8. When treatment is completed, continue to Step 10.




At completion of treatment, press this button to enter Standby Mode (disable laser).

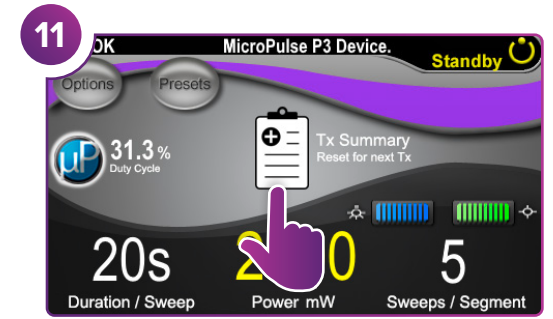


Press to view treatment summary.

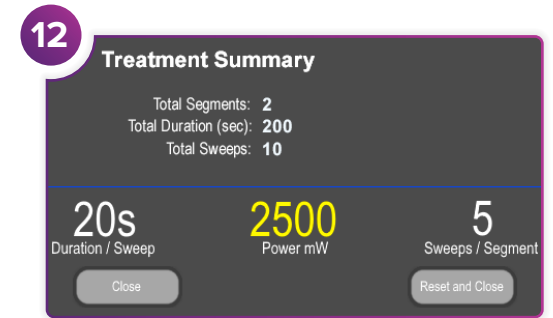

Review treatment summary.

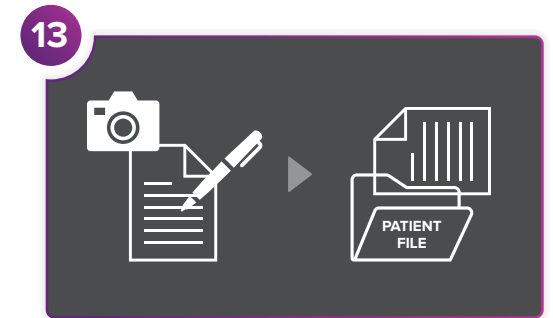

Log treatment information.

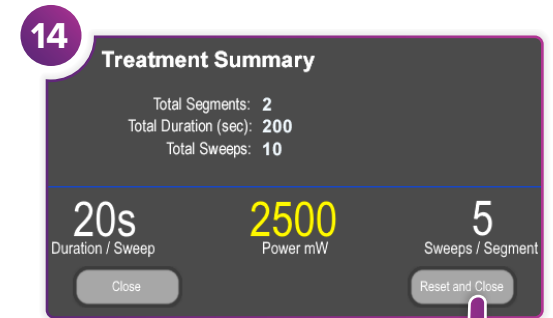

Reset for next treatment.



Iridex • info@iridex.com • iridexglaucoma.com
 © 2022 Iridex. All rights reserved. Iridex, the Iridex logo, MicroPulse, MicroPulse P3, and Cyclo G6 are registered trademarks of Iridex.
 88305-EN.A 02.2022



CE 2797

EC REP

Emergo Europe
 Prinsessegracht 20
 2514 AP The Hague
 The Netherlands

